

Low Water Pressure Issue

Rev 1.3 Oct 2013

SIRONA DENTAL Inc. is investigating reports of low water pressure issues with CEREC MC and CEREC MC X milling units. Our current solution to this problem is to equip all CEREC MC and MC X units with CEREC MC XL water tanks (horizontal water filter).

MC XL Water Tank P/N

SIRONA will be factory equipping CEREC MC and CEREC MC X milling units with the MC XL style water tank and filter starting with S/N listed below.

CEREC MC starting S/N 202101, CEREC MC X starting S/N 231501

For customers in the field, Patterson will

Use replacement water filter P/N 6129519 for these MC XL tanks as the original filter for the MC /MC X will not work.

Note: SIRONA will be factory equipping CEREC MC and CEREC MC X milling units with the MC XL style water tank and filter starting with S/N listed below.

CEREC MC starting S/N 202101, CEREC MC X starting S/N 231501

